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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

DESCRIPTION

Memory pillow is an innovative solution designed with the comfort and health of users in mind. Made of memory foam, it perfectly adapts to the shape of the head, neck and nape, providing optimal support during sleep. Its ergonomic shape supports the natural curvature of the spine, which helps reduce muscle tension and improve the quality of rest.

APPLICATION

- Relaxation and regeneration
- For people with health problems
- Travel
- For everyone

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

TECHNICAL DATA

- Dimensions: 45 cm x 30 cm x 11cm

SAFETY INSTRUCTIONS

- Memory pillow is designed to support your neck and shoulders while you sleep or relax. Do not use it in a way that may cause improper support for your body.
- If you experience any discomfort, pain or muscle tension during use, discontinue use of the pillow immediately and consult your doctor or physiotherapist.
- If you have cervical spine conditions, chronic neck pain or other health problems, consult your doctor before using the Memory Pillow.
- Do not squeeze or crush the pillow excessively as this may damage the structure of the memory foam.
- The pillow is not a toy. Children should only use it under adult supervision.
- Memory foam material is not heat resistant. Keep the pillow away from open flames, radiators and other heat sources.
- Foam is sensitive to moisture. Avoid using the pillow in areas with high humidity or without adequate ventilation.
- Gradually accustom your neck and nape to the pillow, especially if this is your first pillow of this type.
- Check the condition of your pillow regularly
- Make sure the memory foam and cover are not damaged or deformed. If necessary, replace the cushion with a new one.
- Do not wash memory foam because water may damage its structure.
- Do not use in positions that restrict breathing.
- Use a pillow in such a way that it does not cause difficulty in breathing, especially when lying down.
- Make sure the materials used to make the pillow are suitable for people with allergies. If an allergic reaction occurs, discontinue use of the product.

STORAGE, CLEANING

- The pillow should be stored in a dry and well-ventilated room to prevent moisture accumulation and mold growth.
- Do not leave the pillow in places exposed to prolonged sunlight as it may damage the structure of the memory foam.
- If the pillow is not used for a long time, store it in its original packaging or in a special cover to protect it from dust and dirt.
- Avoid storing the pillow near radiators, stoves or other heat sources that may affect the durability of the foam.
- Store the pillow in its natural shape to avoid deformation of the memory foam.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

The product meets European and national requirements on the safety of devices and products.
We reserve the right to make changes to text, design and product data without notice.

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